

rapidex®

Alkaline cleaning solution for surgical instruments

PRODUCT DATA SHEET

- A surfactant blend to dislodge proteinaceous and fatty deposits.
- Medical cleaner for rapid removal of biological matter to prevent re-deposition.

APPLICATION

Recommended for cleaning of surgical instruments, anaesthesia equipment and all receptacles contaminated with biohazardous materials. Rapidex® is suitable for use in ultrasonic cleaning machines and manual cleaning.

- Safe for use on stainless steel, metals, glass and plastics.
- Effective on dried-on deposits of blood and body fluids.
- Ideal for pre-cleaning of soiled equipment in preparation for disinfection and sterilisation.
- Environmentally friendly – readily biodegradable components.

DIRECTIONS FOR USE

- Dissolve one sachet or two scoops in five litres of warm water. The ideal operational temperature is 50°C (can use 20°C–70°C) for a minimum of five minutes.
- For the most effective cleaning, soak surgical and other equipment in Rapidex® as soon as possible after use.
- Remove instruments when practical – either place into a bowl for soak and rinse cleaning or an ultrasonic cleaning machine.
- Disinfect instruments and equipment as necessary.

SPECIFICATION

Composition: a powder mixture of biodegradable alkyl sulphates, inorganic surfactants, dye and alcohol

Appearance: powder

Colour: pale blue

Odour: odourless

Bulk density (tapped): 1.00–1.20

pH value: 10–11 in 1% solution @ 20°C

Miscibility: totally miscible in water (flocculation may occur <150°C)

Pack sizes: bulk powder 2.25kg; Soluble sachets 50 × 28g

STORAGE

Recommended storage temperature: room temperature, keep dry

Shelf life: 5 years

PACKAGING

Bulk powder

2.25kg carton with measure – **order code: rx225**

Outer box contains 6 × 2.25kg cartons – **order code: rx6225**

Soluble sachets

0.6 kg inner carton contains 50 × 12g water soluble sachets – **order code rx250**

Outer box contains 12 × 0.6kg cartons (600 sachets) – **order code rx1250**

1.4 kg inner carton contains 50 × 28g water soluble sachets – **order code rx150**

Outer box contains 6 × 1.4kg cartons (300 sachets) – **order code rx650**

The information contained in this data sheet is believed to be accurate, but customers are advised to ensure that the product is suitable for their own purpose before they use it.

Rapidex® Limited, PO Box 53981, London SW15 6TT, United Kingdom

T +44 (0)870 162 0788 E sales@rapidex.co.uk W rapidex.co.uk

Rapidex® is a registered trademark.