KNEE WRAP APPLICATION: For best results, please read entire application steps before using. View application video at www.STAYONS.com.

Using a STAYONS Poultice Leg Wrap, fold wrap loosely and submerge in hot or cold water as needed, for 15 to 20 seconds.

Poultice rapidly absorbs and holds water. Remove wrap and allow excess water to drain back into pail.

Once applied, poultice material will fill pockets evenly. Apply poultice wrap from back of the knee, placing 1 - 2 filled pockets above the knee joint. Wrap towards front of the knee.

Keep poultice wrap snug, while smoothing and molding around knee and leg area.

Apply Knee Wrap Support over the knee, with about 1 inch of poultice wrap still visible at top.

Snug closure straps across front of leg only. Secure on side of leg, from front to back, so strap ends point towards back of horse.

Snug lower section of Knee Wrap Support material around bottom of the poultice wrap.

Apply a stable bandage, below the knee, to further secure and support the poultice wrap. Recheck within 12-18 hours.

Finished application.

IMPORTANT: Closure straps must be pulled **ACROSS THE FRONT OF THE KNEE AND SECURED.**

DO NOT APPLY ANY TENSION ON OR ACROSS THE BACK OF THE KNEE WHEN SECURING A SNUG FIT.

Initial placement of wrap support will vary depending on the size of the leg itself.

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