



STAYONS hoof wraps incorporate a patent pending design. A specialized fiber paper matrix holds the poultice within 12 quilted pockets. The prefilled poultice consists of bran + epsom salts blended in a proprietary ratio with an organic base to provide an extraordinary contact poultice gel when activated by water. The poultice stays in contact with the hoof & coronary band – exactly where it's placed.

USES: Bran based poultice has traditionally been used for abscesses or suspected abscesses. Consider using this type of wrap for maximum drawing power for potential abscess situations. Also use for general soreness, punctures, inflammation, bruised soles and following heavy work or surgery. Combine with other topical therapy, poultice or antibiotics when required.


APPLICATION: Clean hoof & remove caulks if possible.

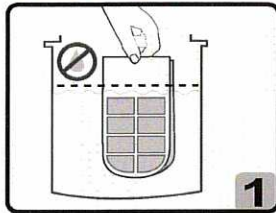


Hold hoof wrap by non-filled ends to keep dry. Submerge remainder into warm or cold water for 15-20 seconds.

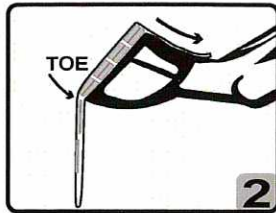

Pick up foot, place section marked "TOE", at the toe. Lay hoof-shaped section across bottom of foot. Cover bulb of heel & extend dry end onto pastern.

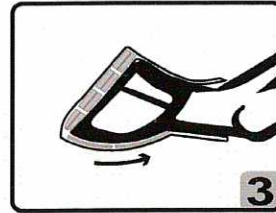

Wrap remaining section onto hoof wall. Cover the coronary band & extend the dry end of wrap onto pastern.

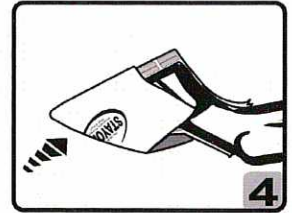

For maximum protection, cover wrap with STAYONS Poultice Boot. Alternatively, use cohesive bandage & duct tape.

Removal: Simply remove within 24 hours to assess. If more poultice time is needed, rewrap for extra poultice time, only if hoof wrap is still wet. Once wrap has dried, adding more water will not reactivate the gel. Replace with fresh wrap. The quilted matrix eliminates poultice build-up on hoof. Minimal to no wash-up required. Easy on water use, easy on hands, easy on drains. Compost, or place used wraps in your regular garbage.

STORAGE: Reseal unused wraps in their original packaging and store in a dry location.

PRECAUTIONS: KEEP AWAY FROM CHILDREN. Plastic bags / sleeves can be dangerous to young children. Contents of wrap not intended for consumption. Avoid breathing dust from dry wraps. Wash hands after handling.

INGREDIENTS: Red wheat bran, Epsom salt, *Cyamopsis tetragonoloba*, fiber paper matrix, non sterile