

# TICKER®

TICK-REMOVER



DOUBLY  
FANTASTICK!



**I'm green™**

Made in Green PE,  
biobased material 100% recyclable.  
100% recycled packaging!



injection



surgery



orthopedics



hospitalization



capture



protection



hygiene



livestock



grooming

**génia**  
the veterinarian's preference

## TIKER®

NEW

**To safely remove ticks! Efficient and quick !**

**Tiker® allows you to remove ticks from animal or people skin. Suitable thanks to the 2 sizes of spoons in the same product which allow to remove easily all type of ticks.**

- Made in Green PE, biobased material 100% recyclable. 100% recycled packaging!
- Fast.
- Painless.
- Hygienic.
- Without leaving the mouth-parts of the tick planted in the skin.
- The spoon allows to hold the tick the time to eliminate it.

Supplied by 24 in a dispenser box.

Code x 24

Model

191061


Tiker®



Watch the video version  
on our Youtube channel !




## DOUBLY FANTASTICK AND ECOLOGICAL!

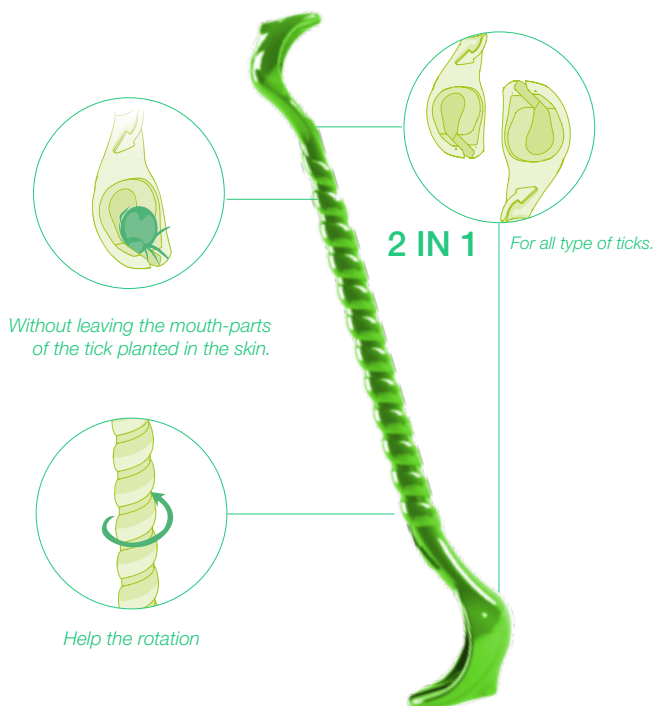


## HOW TO USE


1. Choose spoon size adapted to the tick.
2. Engage the spoon on the parasite side until grasped it.
3. Turn the Tiker® without pulling, until the tick detaches by itself.

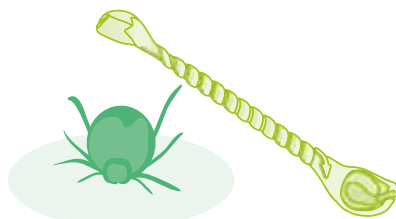

1.

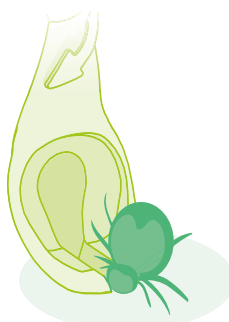

2.

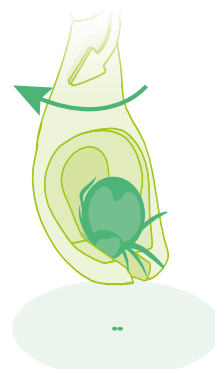

3.

**Caution :** Infectious agents can be transmitted by the tick like Piroplasmosis (dogs and horses), Ehrlichiosis, Lyme (dogs) before to its removal. Lyme is also a zoonosis which mean that people can be infected.

Tiker® cannot help prevent the development of these diseases. Always consult your veterinarian or doctor if symptoms occur.



HOSPITALIZATION