


Coag&Cut M120

120 W Monopolar & Bipolar Electrosurgery unit

Perfect for daily small to medium surgeries



Cut, coagulation and micro-coagulation

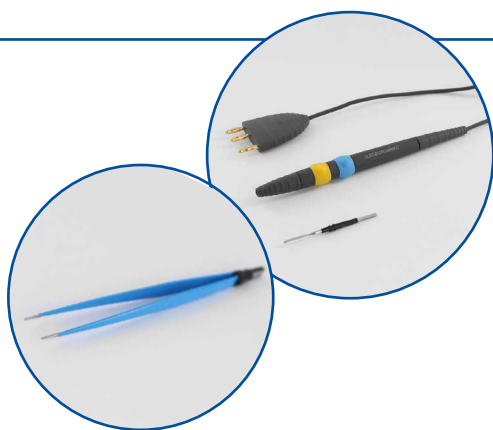

• Perfect for daily small and medium surgeries

The high-performance **Coag&Cut M120** electrosurgery unit has a number of operating modes:

Cutting:

- ▶ CUT = pure Cut
- ▶ BLEND = Cut with Coagulation

Coagulation:

- ▶ FORCED = Superficial Coagulation
- ▶ SOFT = Deep Coagulation
- ▶ BIPOlar = Standard Bipolar Coagulation

• Reliability

The digital power setting display and the continuous monitoring of the operational function by micro processor guarantee an absolute reliability of working conditions.

Certified compliant MDD/93/42 EC Class: IIb



• Safety



The connection to the neutral plate is continuously monitored.

The monopolar handpiece, when used without a footswitch, allows to choose and activate either the Cut Mode (yellow button) or the Coagulation Mode (blue button). Coupled with the footswitch, it allows to choose the function which is activated only by the footswitch. Surgical procedures can be done safely.


Compliant with safety standards EN60601-1-1 / EN60601-1-2 / EN60601-2-2

• Efficient

Maximum Output Power:

- | | |
|------------|-----------|
| ▶ CUT: | 120W-250Ω |
| ▶ BLEND: | 90W-200Ω |
| ▶ FORCED: | 80W-150Ω |
| ▶ SOFT: | 60W-100Ω |
| ▶ BIPOlar: | 40W-100Ω |

Working frequency:	600 kHz
Main frequency:	50-60 Hz
Electrical input power:	300 VA
Size & Weight:	25 x 10 x 29 cm - 5 kg

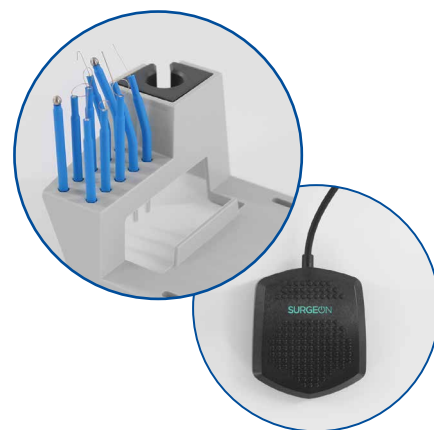

The Coag&Cut M120 package includes:

- ▶ A monopolar handpiece with a straight blade electrode
- ▶ A set of 10 additional electrodes (needles, loops, balls) + support
- ▶ A patient return electrode with cable
- ▶ A bipolar handpiece with cable
- ▶ A single footswitch

All the accessories are autoclavable to ensure durability and good hygiene



**Mano
Médical**
by **génia**

15, rue du Noroît
22100 Taden - France
Tel. +33 (0)2 96 85 86 76

welcome@manomedical.com

www.manomedical.com