KD-PENOFINE

Pen needles

List of compatible pens:

Owen Mumford	AutoPen®	✓
DONGBAO	YpsoPen®	✓
Novo Nordisk	NovoPen® 3	✓
	NovoPen® 4	✓
	NovoPen® Junior	✓
	NovoMix®30 FlexPen®	✓
Ypsomed	YpsoPen®	✓
BD Medical	Puregon Pen®	✓
Bioton	GensuPen2	✓
	GensuPen Improve	✓
Eli Lilly	Luxura HumaPen®	✓
	HumaPen®	✓
	HumaPen® Ergo	✓
	Saline KwikPen™	✓
	Luxura HD HumaPen®	✓
Sanofi Aventis	HumaPen® Ergo II	✓
	OptiPen® Pro 1	✓
	TactiPen®	✓
Lantus	Sanofi Aventis Lantus	✓
	Lantus® SoloStar®	✓
Gan & Lee	GanLee Pen®	✓

Date of last revision: 2019-09-17 50210

EN Do not use if package is damaged

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EN

Pen needles

KD-PENOFINE sterile pen needles for single use are needles intended to be connected to compatible Insulin pens for Insulin administration within the frame of a diabetes mellitus treatment regimen.

Instructions for use:

- Check the compatibility between the pen needle and the Insulin pen – a list of compatible pens is provided in the table.
- Check the product and package integrity prior to use and also the expiry date.
- Do not use if the individual package is already opened or damaged as the device is no longer sterile in this case.
- Do not use after the expiry date.
- Use the product immediately after opening the individual package.
- Connect the pen needle to the Insulin pen.
- Make sure that the Insulin medication comes out properly before each injection. Always keep backup pen needles as a replacement.
- Use the technique recommended by your doctor for administration. Once the pen needle is inserted, press the injection button all the way down, being careful not to change the angle of the Insulin pen. If, after the needle is removed from the skin, you notice medication dripping, keep the needle inserted longer the next time, at the same dosage.
- When the injection is completed, remove the needle from the pen and discard it in an appropriate container in accordance with applicable regulations and guidelines.

Warnings/Notes:

- Consult your doctor regarding the most suitable needle length and injection technique to be used.
- Always follow your doctor's instructions.
- People with reduced mental/physical capacities should be assisted while using the device.
- The pen needle is for single use only – always use a new needle for each injection and discard it directly afterwards. Re-use of this single-use device might lead to infections as a used needle is no longer sterile. Furthermore, injections with an already used needle might be more painful as the lubricant is removed and the tip shape is deformed.
- Never store the Insulin pen with a mounted pen needle as the mounted needle provides an open channel to the Insulin medication inside the cartridge - the Insulin medication may be altered (e.g. crystallization) or contaminated, air may enter into the cartridge and dosage might become inaccurate.

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