

# Bellows Suction System - G050

## INSTRUCTIONS

1. Before use, inspect grenade. Do not use if damaged.
2. Insert step adapter through the blue cap FULLY into silicone sleeve within the hub. Tighten the cap to lock step adapter into place.
3. Connect the step adapter to the drain tubing.  
**NOTE:** SMALL BORE DRAINS will require the FUNNEL ADAPTER.
4. Fully compress the grenade to expel all of the air.
5. Attach the drain tubing to the grenade with the bellows compressed. Suction should immediately occur. If not, inspect all connection points.
6. Secure the grenade to the patient per hospital protocol.



### LARGE BORE DRAINS

*Jackson Pratt Drains (all sizes)  
Blake Drains (all sizes)  
Spiral Drains 12Fr, 15Fr, & 19Fr*

Attach drain tubing to the step adapter.



### SMALL BORE DRAINS

*Multipurpose Drains 8Fr & 10Fr  
Spiral Drains 7Fr & 10Fr*

Attach the funnel adapter to the step adapter before connecting drain tubing.